meomask

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ENGLISH

It is necessary that all statements and instructions of this user manual are followed and fulfilled.

1. Product Overview

The meomask is a surgical face mask of type II according to EN 14683: 2019 + AC: 2019. It consists of three layers of physiologically harmless non-woven polypropylene, with the middle layer giving the meomask its good filter properties thanks to its very fine, meltblown microfibres (meltblown process). When used correctly, the mask completely covers the wearer's nose, mouth and chin area and can be fixed

behind the ears with elastic ear loops. A nasal clip enables an individual fit to the face of the user.

2. Intended Use

The meomask serves as a physical barrier that protects others in particular from the transmission of pathogens in respiratory secretions, liquids or other deposits left by the wearer. In addition, the mask can protect the wearer from unconscious contact of the mouth and nose with possibly contaminated hands and from splashes of liquid.

Due to the size of the masks, they are suitable for use by children aged 8 and over, adolescents and adults of all ages.

The masks are intended for single use.

In certain cases, the meomask should only be used after careful consideration:

- if case of a known allergy,
- for people who cannot wear masks due to medical or psychological impairments, and
- during intense physical exertion.

3. Notes

The masks are not suitable for use with harmful gases and vapors, in hypoxic environments, for underwater operations or for firefighting.

Notes related to Covid-19:

Note that the meomask alone does not protect you from the infection with the novel SARS-CoV-2 virus. Therefore, adhere to the general recommendations of the federal government: keep your distance, observe your hygiene status, wear a mask.

4. Handling Recommendations

Wash or disinfect your hands with soap and water before removing the mask from its packaging. Make sure that there is no obvious damage on either side of the mask. Determine which side of the mask is the top. The side of the mask that has a stiff, pliable edge is the top and can be adjusted to the shape of your nose. Determine which side of the mask is the front. The folds on the front of the mask point downwards. Hold the mask by the ear loops and place a loop around each ear. Pull the meomask over your nose, mouth and chin area and adjust the noseband to the shape of your nose.

Make sure that the meomask fits close to your face. The looser the mask, the greater the proportion of air that flows past the meomask when breathing. Do not touch the front of the meomask when wearing it as it is contaminated. Change the mask when it is wet. We recommend changing the mask every 4 hours and when necessary. There are special regulations for health facilities as to when and how often a mask must be changed.

The meomask is intended for single use. Dispose of them quickly after use.

When removing the meomask, you should ensure that you do not touch the front of the mask but remove it via the ear loops. Wash your hands appropriately with soap and water.

5. Traceability

In order to assure immediate action in the event of changes or recalls, the person responsible for the material store must keep a record of the devices in stock, including the article no. (REF) and batch no. (LOT).

6. Packaging & Sterility

The meomask is packed non-sterile in a dispenser box.

7. Storage Conditions

The meomask must be stored in its original packaging, in a clean and dry environment, protected from vermin, extreme temperatures and moisture. The storage conditions are still indicated on the packaging.

8. Directive Compliance

This medical product is CE-marked in accordance with the Council Directive 93/42/EEC on medical devices (MDD).

9. Disposal

The product can be thermally recycled or disposed of in landfills without releasing toxic substances.

Separate regulations apply to health facilities. In the case of contaminated products, the type and extent of contamination determine the disposal, whereby the applicable national laws and regulations must be observed.